**B&B**  
MEDICAL TECHNOLOGIES

**StabilTube™**  
Latex Free Long Term  
Endotracheal Tube Holder

REF # 11060  
Patented

Contents: 1 UNIT

— Not Sterile —

Read and Understand Complete  
Instructions Before Using.  
Disposable - Single patient use only.

**Caution: Federal (USA) Law restricts this  
device to sale by or on order of a physician.**

Manufactured for:  
**B&B MEDICAL TECHNOLOGIES**  
by:

**INTEGRA BIOTECHNICAL, LLC**  
2755 Dos Aarons Way, Suite B  
Vista, CA 92081 U.S.A.  
**Phone +1.760.929.9972**  
**Fax +1.760.929.9953**

[www.BandB-Medical.com](http://www.BandB-Medical.com)

EU Representative:  
Scanlan Group B.V.  
Postbus 75664  
1118 ZS Schiphol Triport  
The Netherlands  
**Phone +31.20.653.0553**  
**Fax +31.20.653.3053**

LOT #

Form: S106U Rev 1 © 2010 All Rights Reserved  
Made in USA Assembled in Mexico

**READ AND UNDERSTAND COMPLETE  
INSTRUCTIONS BEFORE USING:**

1. Clean and dry the patient's skin and E.T. Tube application area. Be sure all moisture, oils and secretions are removed. Apply the supplied skin preparation solution to both the skin and E.T. Tube application areas and fan dry for 25 seconds. Release liner from Tee tab (Fig. A).

2. Place the retaining unit over the head of the endotracheal tube (Fig. B). Press the adhesive Tee Tab onto the ET Tube and secure the retaining unit tight, 1/2 inch above the lip line identification marking on the tube (Fig. C). Do not place anything between retaining unit and E.T. Tube.

3. Peel the remainder of backing from the face piece and secure to skin (Fig. D). Place over the lip for oral intubation (Fig. E) or over the bridge of the nose for nasotracheal intubation.

4. Slide the VELCRO® brand neckband under patient's neck and release hook and loop strap.

5. Thread neckband strap through the D ring and fasten. Unit should be snug around neck (Fig. F).









6. After confirmation of the proper tube placement, cut the excess tab from the Releasable Cable Tie 1/2 inch above retaining unit (Fig. G). Place Blue Cap on cut end of Cable Tie. Periodically inspect StabilTube for loss of adhesion on face or E.T. Tube and Cable Tie for secure retention. Replace immediately if either occurs.

To remove or reposition, remove Blue Cap, deflect Cable release and push excess tab through retaining unit (Fig. H).

An instructional application video can be found at [www.bandb-medical.com/content/11060\\_vid.htm](http://www.bandb-medical.com/content/11060_vid.htm)

VELCRO® is a Registered Trademark of VELCRO Industries, B.V.

**CAUTION: NEVER use Tincture of Benzoin or other adhesive enhancing products with this product. Always use the Skin barrier provided. Single Patient Use Only • Disposable • Not Sterile • DO NOT REUSE • DO NOT REPROCESS •**

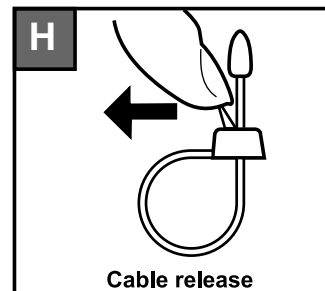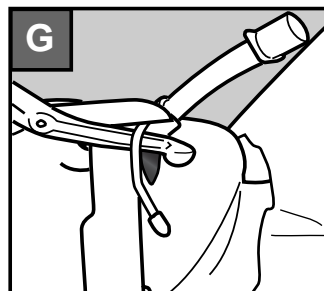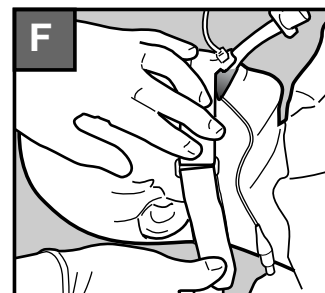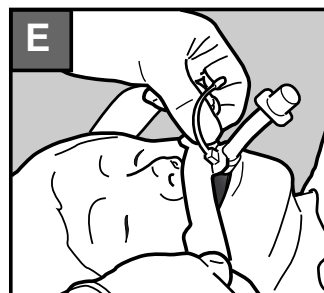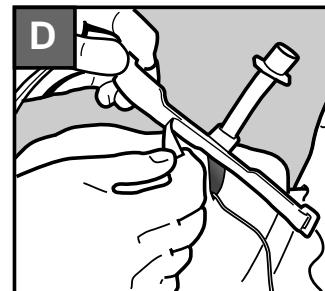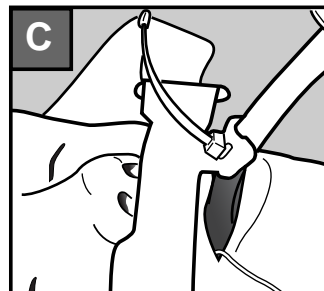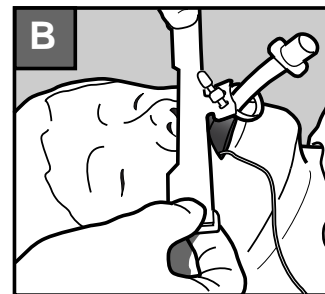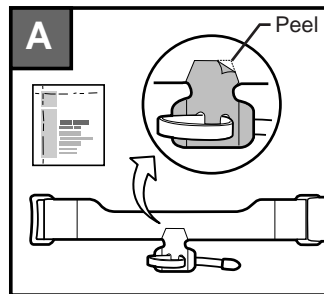Spanish, German, French on reverse side

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VELCRO® est une marque de commerce de VELCRO Industries, B.V.
VELCRO® ist eine eingetragene Marke von VELCRO Industries, B.V.
VELCRO® es una marca comercial registrada de VELCRO Industries, B.V.