# B&B

MEDICAL TECHNOLOGIES

## E.T. Tape II™

Endotracheal Tube Holder  
With Releasable Cable Tie

REF # 11010

Contents: 1 UNIT

— Not Sterile —

Read Complete instructions in  
package insert before using.  
Disposable - Single patient use only.

Caution: Federal (USA) Law restricts this  
device to sale by or on order of a physician.

MFG for  
**B & B MEDICAL TECHNOLOGIES**  
by:  
**INTEGRA BIOTECHNICAL, LLC**  
2755 Dos Aarons Way, Suite B  
Vista, CA 92081 U.S.A.  
Phone +1.760.929.9972  
Fax +1.760.929.9953  
[www.BandB-Medical.com](http://www.BandB-Medical.com)

EU Representative:  
Scanlan Group B.V.  
Postbus 75664  
1118 ZS Schiphol Triport  
The Netherlands  
Phone +31.20.653.0553  
Fax +31.20.653.3053

LOT #

S 101U Rev 1.3  
Made in USA

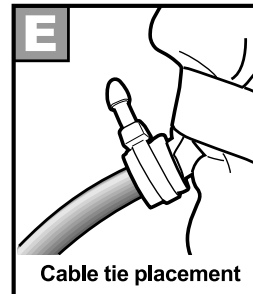
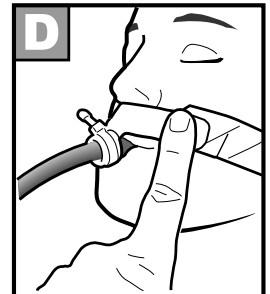
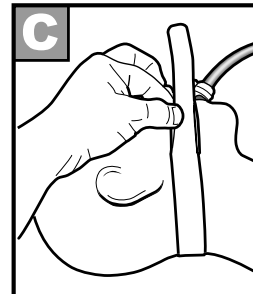
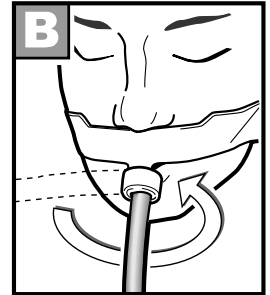
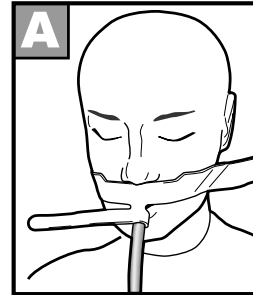
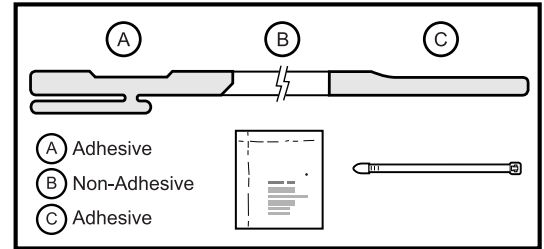
© 2010 All Rights Reserved  
Assembled in Mexico

### READ COMPLETE INSTRUCTIONS BEFORE USING:


1. Clean and dry the patient's skin and E.T. Tube application area. Be sure all moisture, oils and secretions are removed.
2. Apply the supplied skin preparation solution to both the skin and E.T. Tube application areas and fan dry for 25 seconds.
3. Peel backing from Part A. Secure the E.T. Tape II™ to the patients skin with the T-Tab directly over the desired E.T. Tube location. (Fig. A).
4. Wrap the short leg of the T-Tab around the E.T. Tube.
5. Tightly wrap the long leg of the T-Tab around the tube overlapping the short leg. This wrap must be tight as to compress the foam, eliminating any gaps which would allow secretions to get between the E.T. Tape II and the E.T. Tube. (Fig. B).
6. Wrap Cable Tie around T-tab, endotracheal tube and Bite Block #11040 or #11160 and pull tight. Cut off excess Cable Tie and place Blue Cap on end of Cable Tie.
7. Pass E.T. Tape II behind the patient's neck, under the earlobes.
8. Peel backing from Part C.
9. While holding Part A with finger, pull the E.T. Tape II taut, overlapping Part A with Part C. (Fig. C).
10. Firmly press the tape for proper adhesion. (Fig. D).
11. To remove Cable Tie, remove Blue Cap and release lock on Cable Tie. (Fig. F)
12. E.T. Tape II may be cut to proper length to fit smaller patients.
13. Periodically inspect E.T. Tape II™ for loss of adhesion on face or E.T. Tube. Replace immediately if this occurs.

**CAUTION:** Single Patient Use Only • Non Sterile  
• Disposable • **DO NOT REPROCESS or REUSE •**

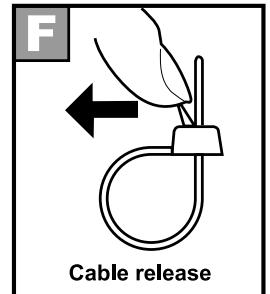Spanish, German, French on Reverse Side

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