B&B

MEDICAL TECHNOLOGIES

E.T. Tape™

Coated E.T. Tube Holder
With Releasable Cable Tie

REF # 11020

Contents: 1 UNIT

— Not Sterile —

Read Complete instructions in
package insert before using.
Disposable - Single patient use only.

Caution: Federal (USA) Law restricts this
device to sale by or on order of a physician.

MFG for

B & B MEDICAL TECHNOLOGIES

by:

INTEGRA BIOTECHNICAL, LLC

2755 Dos Aarons Way, Suite B

Vista, CA 92081 U.S.A.

Phone +1.916.331.5221

Fax +1.916.331.0161

www.BandB-Medical.com

EU Representative:
Scanlan Group B.V.
Postbus 75664
1118 ZS Schiphol Triport
The Netherlands
Phone +31.20.653.0553
Fax +31.20.653.3053

LOT #

S 102U Rev 1
Made in USA

© 2009 All Rights Reserved
Assembled in Mexico

READ COMPLETE INSTRUCTIONS BEFORE USING:

1. Clean and dry the patient's skin and E.T. Tube application area. Be sure all moisture, oils and secretions are removed.

2. Apply the supplied skin preparation solution to both the skin and E.T. Tube application areas and fan dry for 25 seconds.

3. Peel backing from Part A. Secure the E.T. Tape™ to the patients skin with the T-Tab directly over the desired E.T. Tube location. (Fig. A).

4. Wrap the short leg of the T-Tab around the E.T. Tube.

5. Tightly wrap the long leg of the T-Tab around the tube overlapping the short leg. This wrap must be tight as to compress the foam, eliminating any gaps which would allow secretions to get between the E.T. Tape and the E.T. Tube. (Fig. B).

6. Wrap Cable Tie around T-tab, endotracheal tube and Bite Block #11040 or #11160 and pull tight. Cut off excess Cable Tie and place Blue Cap on end of Cable Tie.

7. Pass E.T. Tape behind the patient's neck, under the earlobes.

8. Peel backing from Part C.

9. While holding Part A with finger, pull the E.T. Tape taut, overlapping Part A with Part C. (Fig. C).

10. Firmly press the tape for proper adhesion. (Fig. D).

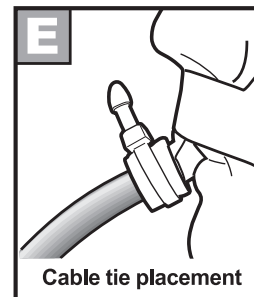
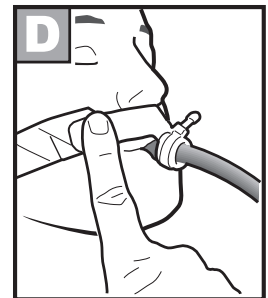
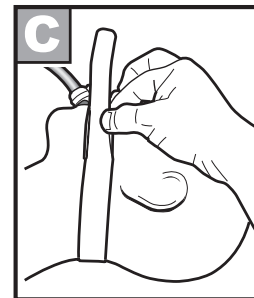
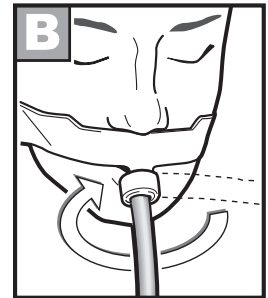
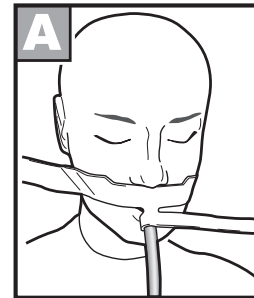
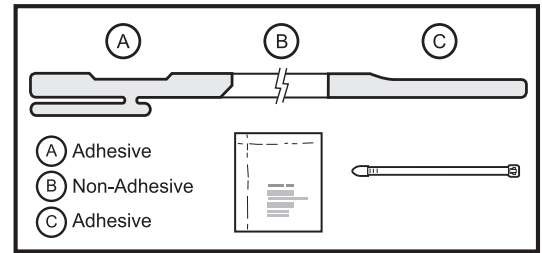
11. To remove Cable Tie, remove Blue Cap and release lock on Cable Tie. (Fig. F)

12. E.T. Tape may be cut to proper length to fit smaller patients.


13. Periodically inspect E.T. Tape™ for loss of adhesion on face or E.T. Tube. Replace immediately if this occurs.

CAUTION: Single Patient Use Only • Non Sterile
• Disposable • **DO NOT REPROCESS or REUSE •**

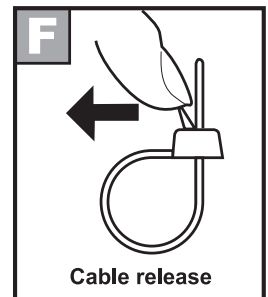Spanish, German, French on Reverse Side

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