

READ AND UNDERSTAND COMPLETE INSTRUCTIONS BEFORE USING:

Use with B&B ET Tape II #11010, ET Tape #11020, StabilTube #11060 or other endotracheal tube securing devices.


1. Secure the endotracheal tube per instructions or institution procedure. Gently pull the pilot balloon tube in a straight line toward the end of the endotracheal tube.
2. Starting with the Universal Bite Block at a slight angle, snap the bite block onto the endotracheal tube. Keeping the bite block at a slight angle, slide it onto the tube. Make sure the pilot balloon tube is in between the Universal Bite Block and the endotracheal tube (see figure B).
3. Pull the Universal Bite Block up until it rests against the endotracheal tube securing tape and rotate the bite block to the lateral position (see figure C).
4. Wrap the elongated tab of the Cable Tie around the securing tape where it wraps around the endotracheal tube and feed through retaining unit (see figure D).
5. After confirming proper tube placement, pull Cable Tie through retaining unit to secure to endotracheal tube and carefully cut the releasable Cable Tie ½ inch above retaining unit. Place Blue Safety Cap securely onto cut end of Cable Tie (see figure E).
6. To reposition or replace Universal Bite Block, carefully remove Blue Safety Cap from Cable Tie, release the locking tab and push excess Cable Tie through retaining unit (see figure F).

WARNING: If releasable Cable Tie is not used, Universal Bite Block must be securely taped to the endotracheal tube. If Cable Tie is not trimmed, remove and discard Blue Safety Cap in proper receptacle. Should endotracheal tube cuff filling problems occur, gently pull pilot balloon tube taut to remove any kinks.

Note: Periodically inspect the Universal Bite Block to assure proper position and function of the bite block. Universal Bite Block should be changed every three days for hygienic purposes.

CAUTION: Single Patient Use Only • Disposable • Not Sterile • DO NOT REUSE • DO NOT REPROCESS •

An instructional application video can be found at www.bandb-medical.com/content/11160_vid.htm

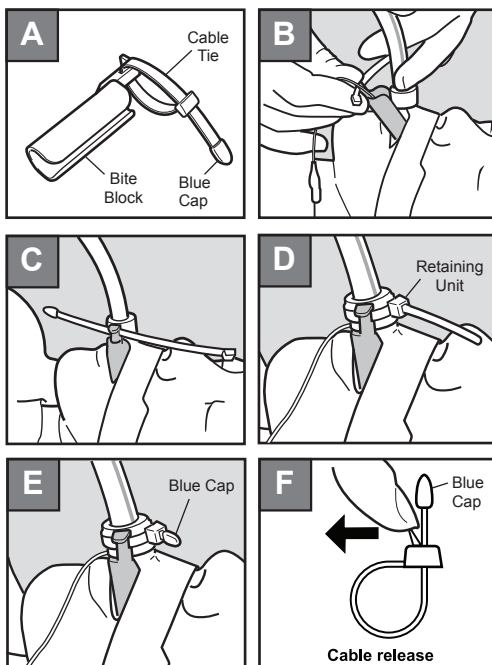

B&B
MEDICAL TECHNOLOGIES

Universal Bite Block™

Latex Free Endotracheal Tube Bite Block
For Tube Sizes 6 mm to 9.5mm

REF # 11160 Patented



Contents: 1 UNIT



— Not Sterile —

Read and Understand

Complete Instructions Before Using.
Disposable - Single Patient Use Only.



Caution: Federal (USA) Law restricts this device to sale by or on order of a physician.

Manufactured for:
B&B MEDICAL TECHNOLOGIES
by:

INTEGRA BIOTECHNICAL, LLC
2755 Dos Aarons Way, Suite B
Vista, CA 92081 U.S.A.
Phone +1.760.929.9972
Fax +1.760.929.9953
www.BandB-Medical.com



LOT #

Form: S116U Rev 1.0
Made in USA

© 2010 All Rights Reserved
Assembled in Mexico