

## UNIVERSAL BITE-BLOCK™ #11170 PEDIATRIC

### INSTRUCTIONS:

Use with E.T. Tape or Adhesive tape products


1. Secure the endotracheal tube per instructions or procedure.
2. With the BITE-BLOCK at a slight angle, slip the bottom portion of the device onto the endotracheal tube behind the tape.
3. Pull the BITE-BLOCK forward until it rests against the tape.
4. Rotate Bite Block to lateral position.
5. Insert the elongated tab of the cable tie tightly around the tape where it circumvents the tube.
6. After tube placement confirmation, cut off remaining cable tie.

**CAUTION:** If the cable tie is not used, the tab must be securely taped to the endotracheal tube.

**NOTE:** Periodically inspect the bite block to assure proper position and function of the device.

Universal Bite Block should be changed every three days for hygienic purposes.

**CAUTION:** Federal (USA) law restricts this device to sale by or on order of a physician. Single Patient Use Only • Disposable • Not Sterile • DO NOT REUSE

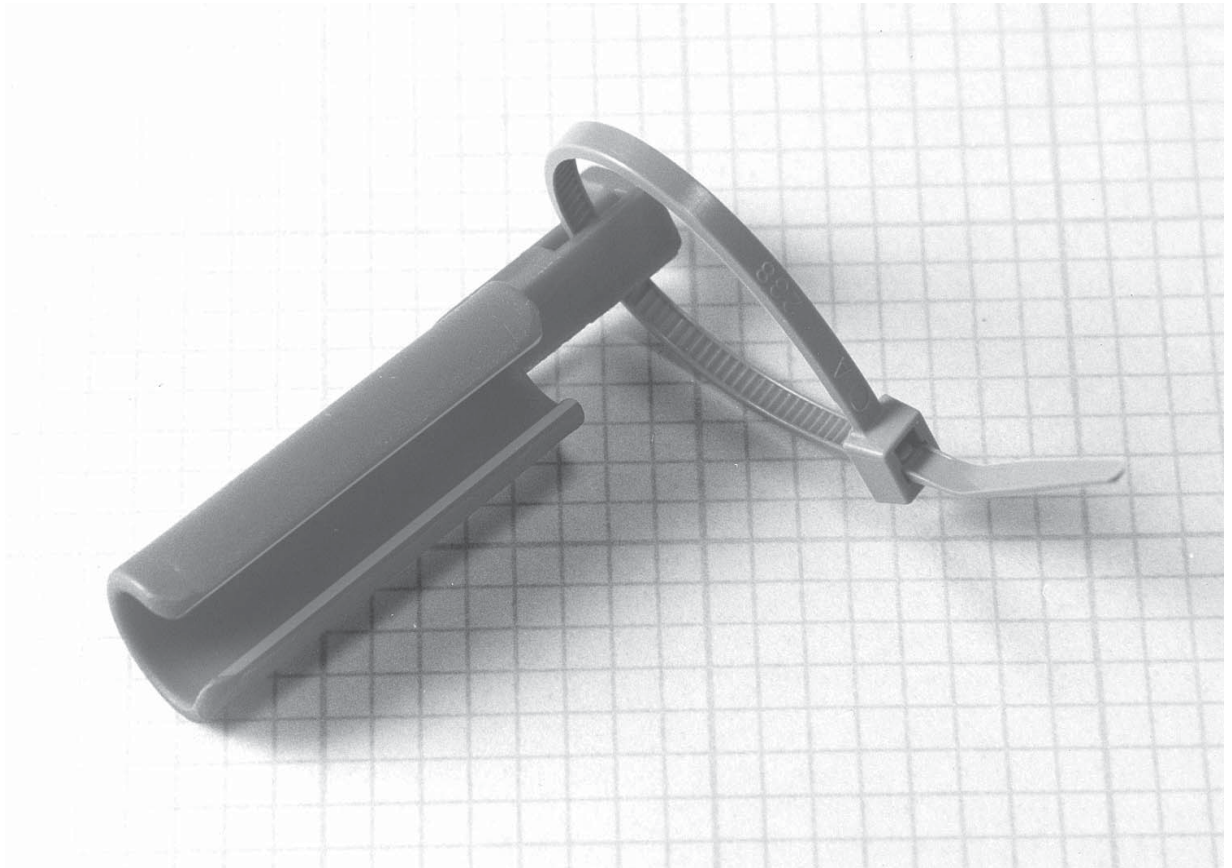

B&B Medical Technologies  
2734 Loker Ave West, Suite M.  
Carlsbad, CA 92010 USA  
www.BandB-Medical.com  
Tel (760) 929-9972  
Fax (760) 929-9953



Reorder # 11170 50 units/case  
Made in U.S.A. - Packaged in Mexico  
Form: S174 Rev. 6.3

© 2009 All Rights Reserved B&B Medical Technologies

EU Representative: Scanlan Group B.V.  
Postbus 75664  
1118 ZS Schiphol Triport  
The Netherlands  
Phone 31(0) 20-653-0553  
FAX 31(0) 20-653-3053