TrachGuard™

Tracheostomy Tube Holder &
Anti Disconnect Device

REF # 11250
Patented

Contents: 1 UNIT

— Not Sterile —

Read and Understand Complete
Instructions Before Using.
Disposable - Single patient use only.

For Use in Hospital and Home Care

Manufactured for:
B&B MEDICAL TECHNOLOGIES
by:

INTEGRA BIOTECHNICAL, LLC
2755 Dos Aarons Way, Suite B
Vista, CA 92081 U.S.A.
Phone +1.760.929.9972
Fax +1.760.929.9953

www.BandB-Medical.com







EU Representative:
Scanlan Group B.V.
Postbus 75664
1118 ZS Schiphol Triport
The Netherlands
Phone +31.20.653.0553
Fax +31.20.653.3053

READ AND UNDERSTAND COMPLETE INSTRUCTIONS BEFORE USING:

1. Wash hands before handling the TrachGuard.
2. Open package, remove collar and separate anti-disconnect Blue O-ring from collar (Figure A).
3. Pass collar behind neck. Holding tracheostomy device securely, thread one of the tapered ends of collar through tracheostomy flange and secure VELCRO® Brand tab to collar (Figure B).
4. Repeat this process on opposite side. Tighten neckband; allow one finger between neck and collar for patient comfort.
5. Open white anti-disconnect loops on Blue O-ring and position Blue O-ring over 15 mm tracheostomy connector (Figure C). Secure Blue VELCRO® wing to each side of trach collar and cut to desired length (Figure D). Do not overlap the Blue straps over white closure tabs of collar as closure may not be optimal.
6. Place white VELCRO® loop over ventilator manifold or suction device.
7. Secure with the white VELCRO® hook by overlapping the loops (Figure E).
8. TrachGuard is properly attached (Figure F). For optimal patient comfort, place Sil.Flex™ Stoma Pad between tracheostomy flange and patient's neck.
9. Inspect collar and anti-disconnect for security as indicated per hospital guidelines or home care protocol.
10. Change when collar is soiled or VELCRO® fasteners do not secure to collar or white loops.
11. Routinely check and readjust collar so that one finger can easily fit between collar and neck and make sure that hook ends of collar are secured to the collar.

Caution: When changing the collar on a patient without changing the tracheostomy tube, be sure to hold the tube in position while changing the collar to prevent accidental decannulation.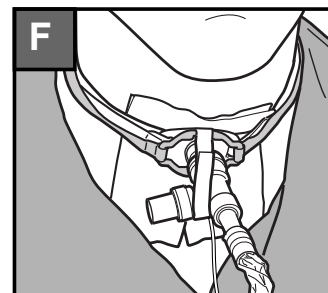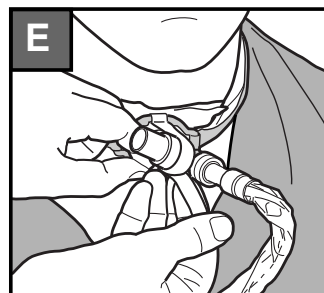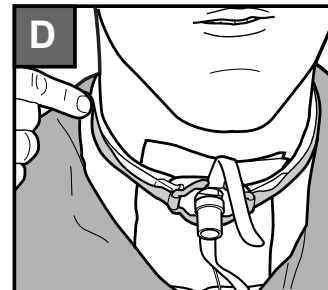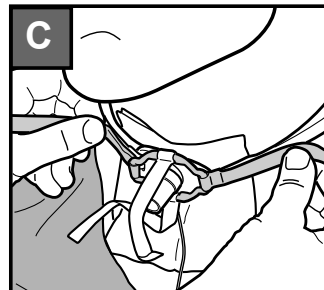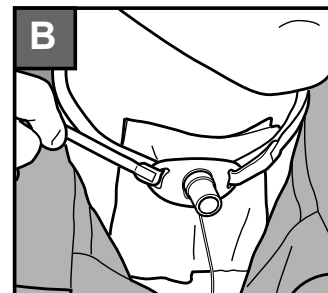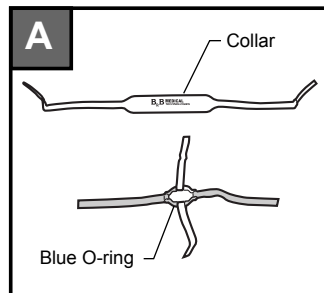

An instructional application video can be found at
www.bandb-medical.com/content/11250_vid.htm

VELCRO® is a Registered Trademark of VELCRO Industries, B.V.

Sil.Flex™ is a Trademark of A Plus Medical

CAUTION: Single Patient Use Only • Disposable • Not Sterile •
DO NOT REUSE • DO NOT REPROCESS •

Spanish, German, French on reverse side

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