
### READ COMPLETE INSTRUCTIONS BEFORE USING:

1. Remove the TrachStay™ from package.
2. Open the VELCRO® loop by separating hook/loop.
3. Place the ring over the tracheostomy tube 15mm connector until it rests on the flange.
4. Starting with one side, release the tracheostomy collar and slip the velcro hook of the collar through the flange and ring and reattach the velcro hook to the collar.
5. Release the tracheostomy collar on the other side and slip the velcro hook of the collar through the flange and ring and reattach the velcro hook to the collar.
6. Attach the ventilator, in-line suction or other supplemental gas device to the tracheostomy tube.
7. Place the VELCRO® straps of the anti-disconnect over the attached device and secure Velcro hook to loop.
8. Periodically inspect the device to assure that it is securely holding attached tubing or suction system.

VELCRO® Brand is a Registered trademark of Velcro Industries B.V.

**CAUTION:** Federal (USA) law restricts this device to sale by or on order of a physician • Single Patient Use Only • Disposable • Not Sterile • DO NOT REUSE •

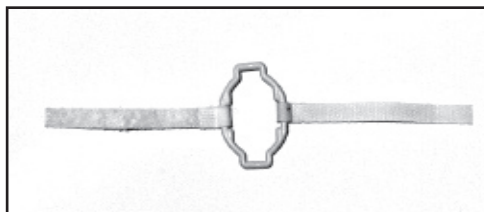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