

Single Pole Clamp

REF 20271




WARNINGS AND CAUTIONS :

- Do not overtighten the clamp as damage to the pole and/or clamp may occur.
- Equipment inserted into the holder should not exceed 3.0 kg.
- Equipment slip mount needs to conform to ISO 19054.

INSTRUCTIONS FOR USE:

1. Turn the knob counterclockwise to open the clamp, obtaining the desired clearance to fit the pole.
2. With the stop block at the bottom (towards the floor), place clamp on the pole.
3. Turn the knob clockwise to tighten the clamp.
4. Slide the equipment mount into the holder channel until it contacts the stop block or is fully seated.

MAJOR COMPONENTS AND INDICATORS:

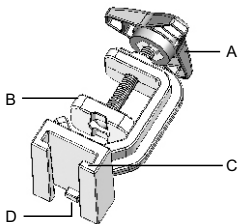

ITEM DESCRIPTION

A	Knob
B	Clamp
C	Holder Channel
D	Stop Block



Designed by A Plus Medical, U.S.A.
A Plus Medical
www.aplus-medical.com
Manufactured by GateMed Corp. In Taiwan
for A Plus Medical.



Distributed by:
B&B Medical Technologies
2734 Loker Avenue West, Suite M, Carlsbad, CA 92010
Tele: +1.760.929.9972 Fax: +1.760.929.9953
www.BandB-Medical.com