



TrachStay™

Sturdy, efficient device stabilizes ventilator connector; stabilizes in-line suction devices

TrachStay provides an easy to use, convenient solution to minimize the potential for inadvertent ventilator circuit disconnects for adult and pediatric patients. TrachStay is a highly efficient and sturdy anti-disconnect device that attaches with all tracheostomy securing devices. TrachStay stabilizes the ventilator connector, in-line suction system or other supplemental gas delivery systems. TrachStay is latex free. Application is easy and secure.

To attach, place the TrachStay ring over the tracheostomy tube until it rests on the flange. Release one side of the tracheostomy collar, slip the hook of the collar through the flange and TrachStay ring, and reattach to the collar. Repeat on the other side of the collar. Attach the ventilator, in-line suction system or other supplemental gas delivery device to the tracheostomy tube. Then place the TrachStay straps over the attached device and secure its hook-and-loop fastener to stabilize the connection.

TrachStay protects against accidental disconnects and cross contamination.

SAFE

- Latex free, hypoallergenic materials
- Enhanced durability for securing the ventilator circuit connection
- Minimizes potential ventilator attachment separation
- Reduces potential exposure to contaminated tubing moisture

COST EFFECTIVE

- Designed for use in critical care, PICU
- Designed for use in sub acute, long term acute, rehab and home care

CONVENIENT

- Minimizes time for setup and connection
- Easy and quick application to all tracheostomy collars

VERSATILE

- Adult and pediatric application
- Can be used with delivery of aerosol and supplemental oxygen therapy
- Anti-disconnect device can be used with all tracheostomy collars

Ordering Information

11270 TrachStay (50/case)

B&B Products are available from

B&B Medical Technologies and finer specialty care distributors.
Visit www.BandB-Medical.com or contact us today at 1.800.242.8778.

