

# Single Pole Clamp

**REF** 25371



## READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE

1. Turn the knob counterclockwise to open the clamp, obtaining the desired clearance to fit the pole.
2. With the stop block at the bottom (towards the floor), place clamp on the pole.
3. Turn the knob clockwise to tighten the clamp.
4. Slide the equipment mount into the holder channel until it contacts the stop block or is fully seated.



## WARNINGS AND CAUTIONS

- Do not overtighten the clamps as damage to the pole and / or clamp may occur.
- Equipment inserted into the holder should not exceed 3.6 kg.
- Equipment slip mount needs to conform to ISO 19054.



**Manufactured for:**  
B&B Medical Technologies  
1954 Kellogg Ave. Ste 100  
Carlsbad, CA 92008 U.S.A  
Phone: +1.760.929.9972  
Fax: +1.760.929.9953  
www.bandb-medical.com

