
E.T. Tape™

Original Endotracheal Tube Holder

Contents: 1 UNIT

REF 11020

LOT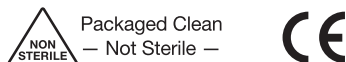

Not Made With
Natural Rubber Latex

43°C
110°F

Caution: Federal (USA) Law restricts this device to sale by or on order of a physician.

Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.bandb-medical.com

EC REP EU Representative:
Scanlan Group B.V.
Postbus 75664
1118 ZS Schiphol Triport
The Netherlands
Phone +31.20.653.0553
Fax +31.20.653.3053

© 2016 All Rights Reserved Patented
Form: S102U Rev 1.5 Made in USA
Assembled in Mexico








READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE

1. Clean and dry the patient's skin and E.T. Tube application area. Be sure all moisture, oils and secretions are removed.
2. Apply the supplied skin preparation solution to both the skin and E.T. Tube application areas and fan dry for 25 seconds.
3. Peel backing from Part A. Secure the E.T. Tape to the patient's skin with the T-Tab directly over the desired E.T. Tube location (see Fig. A).
4. Wrap the short leg of the T-Tab around the E.T. Tube.
5. Tightly wrap the long leg of the T-Tab around the tube, overlapping the short leg. This wrap must be tight to compress the foam, eliminating any gaps which would allow secretions to get between the E.T. Tape and the E.T. Tube (see Fig. B).
6. Wrap Cable Tie around T-tab, endotracheal tube and Bite Block #11040 or #11160 and pull tight. Cut off excess Cable Tie and place Blue Cap on end of Cable Tie.
7. Pass E.T. Tape behind the patient's neck, under the earlobes.
8. Peel backing from Part C.
9. While holding Part A with finger, pull the E.T. Tape taut, overlapping Part A with Part C (see Fig. C).
10. Firmly press the tape for proper adhesion (see Fig. D).
11. To remove Cable Tie, remove Blue Cap and release lock on Cable Tie (see Fig. F)
12. E.T. Tape may be cut to proper length to fit smaller patients.
13. Periodically inspect E.T. Tape for loss of adhesion on face or E.T. Tube. Replace immediately if this occurs.

CAUTION: Single Patient Use Only • Non Sterile
• Disposable • DO NOT REPROCESS or REUSE

Spanish, German, French on reverse side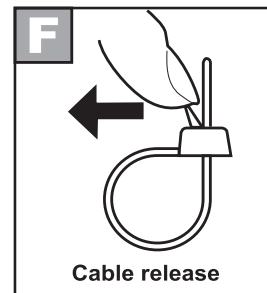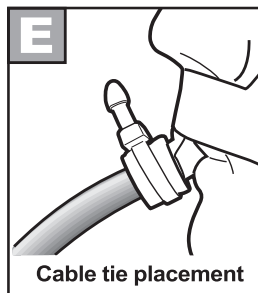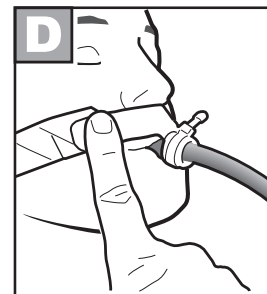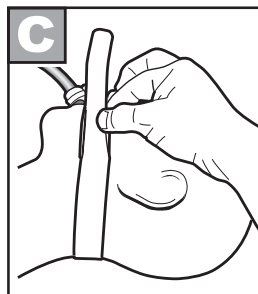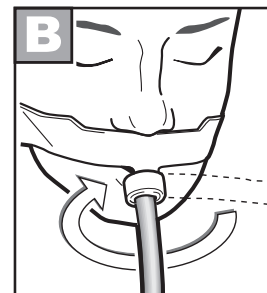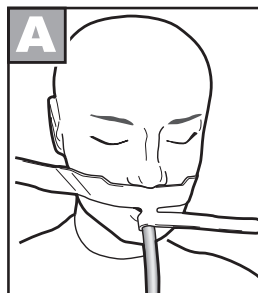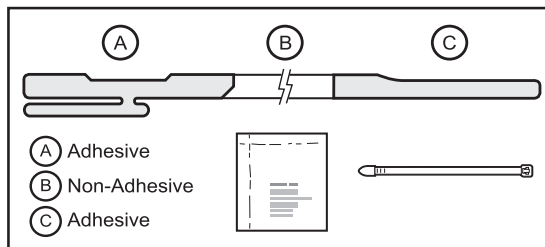

An instructional application video can be found at
www.bandb-medical.com/et-tape

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