READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE







1. Secure the endotracheal tube per instructions or institution procedure. Gently pull the pilot tube in a straight line toward the end of the endotracheal tube.
2. Starting with the Bite-Proof Bite Block at a slight angle, snap the back of the Bite-Proof Bite Block onto the endotracheal tube. Keeping the Bite-Proof Bite Block at a slight angle, slide it onto the tube. Make SURE the pilot balloon is between the Bite-Proof Bite Block and the endotracheal tube (Figure B).
3. Keeping the pilot balloon tube taut, gently slide the Bite-Proof Bite Block down the tube until the wing is at lower lip.
4. Slowly rotate the Bite-Proof Bite Block until the wing is positioned laterally to the tube (Figure C). Peel backing from the wing and secure it to the cheek. Keeping the pilot balloon taut, position the Bite-Proof Bite Block to the proper depth (Figure D).
5. Secure the tab of the Bite-Proof Bite Block to the endotracheal tube with adhesive tape (Figure E). If using the B&B tube holders, secure tube according to direction and overlap tab with tape. Note: Tab must be securely taped to the endotracheal tube (Figure F).
6. Periodically inspect the Bite-Proof Bite Block for proper positioning and function. Replace immediately for loss of adhesion to the face or the E.T. Tube.
7. Bite-Proof Bite Block should be changed every two days for hygienic purposes.

CAUTION: NEVER use Benzoin or other adhesive-enhancing products in conjunction with this product. Should cuff filling problem occur, gently pull the pilot balloon tube taut to remove any kinks. NEVER remove the wing from the Bite Proof Bite Block.

CAUTION:

- Single Patient Use Only
- DO NOT REUSE
- DO NOT REPROCESS
- Not Sterile
- Disposable

An instructional application video can be found at www.bandb-medical.com/bite-proof-bite-block

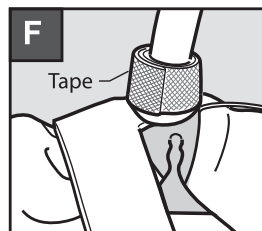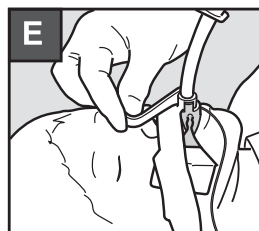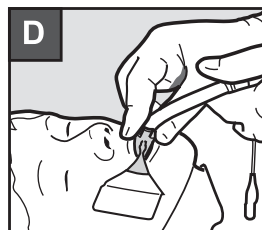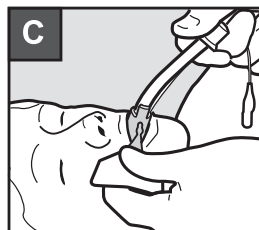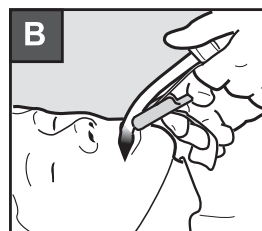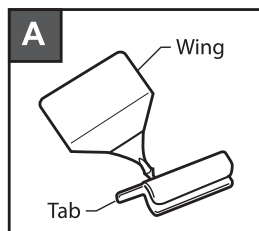BB Medical Technologies
www.bandb-medical.com

Bite-Proof Bite Block™

Latex-Free Endotracheal Tube Bite Block

For Tube Sizes 6 mm to 9.5mm

Contents: 1 UNIT


REF 11040


LOT

Not Made With
Natural Rubber Latex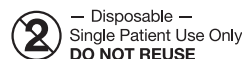

CAUTION: Federal law restricts this device to sale by or on order of a physician.

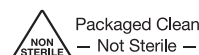Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.bandb-medical.com

EU Representative:
Scanlan Group B.V.
Postbus 75664
1118 ZS Schiphol Triport
The Netherlands
Phone: 31(0) 20-653-0553
Fax: 31(0) 20-653-3053

Patented
Assembled in Mexico
Form S104U Rev 1.2
© 2014 All rights reserved.

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