READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE

Use with B&B ET Tape II #11010, ET Tape #11020, StabilTube #11060 or other endotracheal tube securing devices.

1. Secure the endotracheal tube per instructions or institution procedure. Gently pull the pilot balloon tube in a straight line toward the end of the endotracheal tube.
2. Starting with the Universal Bite Block at a slight angle, snap the bite block onto the endotracheal tube. Keeping the bite block at a slight angle, slide it onto the tube. Make sure the pilot balloon tube is in between the Universal Bite Block and the endotracheal tube (see figure B).
3. Pull the Universal Bite Block up until it rests against the endotracheal tube securing tape and rotate the bite block to the lateral position (see figure C).
4. Wrap the elongated tab of the Cable Tie around the securing tape where it wraps around the endotracheal tube and feed through retaining unit (see figure D).
5. After confirming proper tube placement, pull Cable Tie through retaining unit to secure to endotracheal tube and carefully cut the releasable Cable Tie 1/2 inch above retaining unit. Place Blue Safety Cap securely onto cut end of Cable Tie (see figure E).
6. To reposition or replace Universal Bite Block, carefully remove Blue Safety Cap from Cable Tie, release the locking tab and push excess Cable Tie through retaining unit (see figure F).


WARNING: If releasable Cable Tie is not used, Universal Bite Block must be securely taped to the endotracheal tube. If Cable Tie is not trimmed, remove and discard Blue Safety Cap in proper receptacle. Should endotracheal tube cuff filling problems occur, gently pull pilot balloon tube taut to remove any kinks.

NOTE: Periodically inspect the Universal Bite Block to assure proper position and function of the bite block. Universal Bite Block should be changed every three days for hygienic purposes.

CAUTION:

Single Patient Use Only • Disposable • Not Sterile
• DO NOT REUSE • DO NOT REPROCESS

An instructional application video can be found at www.bandb-medical.com/universal-bite-block-adult/

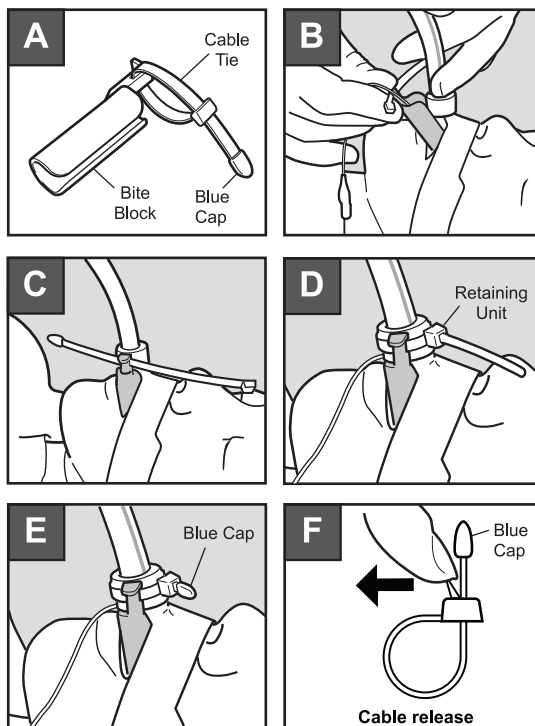Medical Technologies
www.bandb-medical.com

Universal Bite Block™

Endotracheal Tube Bite Block

For Tube Sizes 6 mm to 9.5 mm

Contents: 1 UNIT

REF 11160


LOT

Not Made With
Natural Rubber Latex

CAUTION: Federal law restricts this device to sale by or on order of a physician.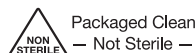

Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A.
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.bandb-medical.com

EC REP

EU Representative:

Scanlan Group B.V.
Postbus 75664
1118 ZS Schiphol Triport
The Netherlands
Phone: 31(0) 20-653-0553
Fax: 31(0) 20-653-3053

Patented
Assembled in Mexico
Form S116U Rev 1.4
© 2015 All rights reserved.

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