
**READ AND UNDERSTAND
COMPLETE INSTRUCTIONS
PRIOR TO USE**

Use with E.T. Tape or adhesive tape products.

1. Secure the endotracheal tube per instructions or procedure.
2. With the Universal Bite Block at a slight angle, slip the bottom portion of the device onto the endotracheal tube behind the tape.
3. Pull the Universal Bite Block forward until it rests against the tape.
4. Rotate bite block to lateral position.
5. Insert the elongated tab of the cable tie tightly around the tape where it circumvents the tube.
6. After tube placement confirmation, cut off remaining cable tie.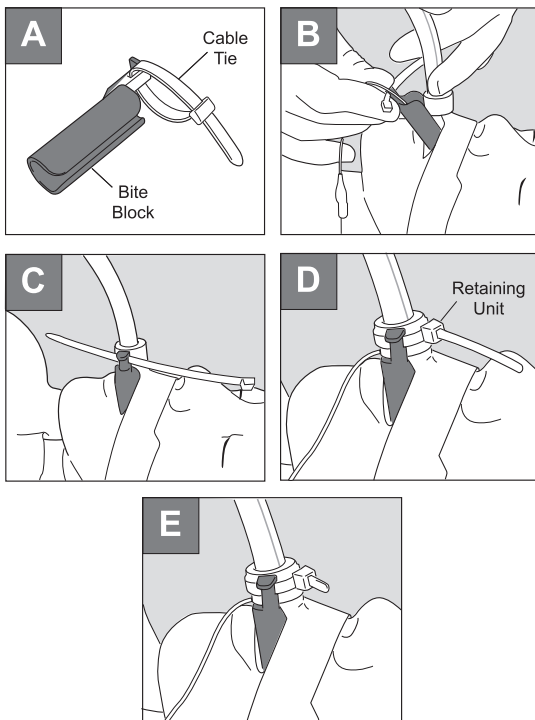

CAUTION: If the cable tie is not used, the tab must be securely taped to the endotracheal tube.

NOTE: Periodically inspect the bite block to assure proper position and function of the device.

The Universal Bite Block should be changed every three days for hygienic purposes.

CAUTION:

- Single Patient Use Only
- DO NOT REUSE
- Not Sterile
- Disposable

An instructional application video can be found at www.bandb-medical.com/content/11170_vid.htm

BB Medical Technologies
www.bandb-medical.com

Pediatric Universal Bite Block™
Endotracheal Tube Bite Block
For Tube Sizes 4 mm to 6 mm

Contents: 1 UNIT

REF 11170

LOT

USE
BY:

43°C
110°F

Not Made With
Natural Rubber Latex

! READ PACKAGE
INSERT FOR
INSTRUCTIONS

2 — Disposable —
Single Patient Use Only
DO NOT REUSE

CAUTION: Federal law
restricts this device to sale by
or on order of a physician.

**NON
STERILE** Packaged Clean
— Not Sterile —

Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.BandB-Medical.com

EC REP **EU Representative:**
Scanlan Group B.V.
Postbus 75664
1118 ZS Schiphol Triport
The Netherlands
Phone: 31(0) 20-653-0553
Fax: 31(0) 20-653-3053

Patented
Assembled in Mexico
Form S117U Rev 1.0
© 2014 All rights reserved.

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