

WRAP-SAFE™

Tames Unruly Hoses, Power Cords &
Communication Cables

Contents: 10 units/Bag

Made in USA

**Installation Instructions:**

1. Place the Wrap-Safe™ on a firm surface with the B&B logo facing down.
2. Lay the hose or cord in between the two holes on the Wrap-Safe™ (fig 1).
3. Place the Wrap-Safe™ strap around the cord with the hook side facing out and snap the male and female halves of the rivet together through the holes (fig. 2).
4. Secure by wrapping the Wrap-Safe™ around the coiled hose or cable (fig. 3).



Figure 1



Figure 2



Figure 3

B&B Medical Technologies
P.O. Box 615, North Highlands, CA 95650-0615 USA

WWW.BANDB-MEDICAL.COM

Phone (916) 331-5221 FAX (916) 331-0161