
# E.T. Tape II™

## Endotracheal Tube Holder With Releasable Cable Tie

Contents: 1 UNIT

**REF** 11010

**LOT**

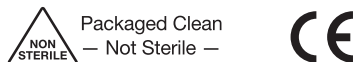**2** — Disposable —  
Single Patient Use Only  
**DO NOT REUSE**

Not Made With  
Natural Rubber Latex

Caution: Federal (USA) Law restricts this device to sale by or on order of a physician.

**Manufactured for:**  
B&B Medical Technologies  
1954 Kellogg Ave. Ste 100  
Carlsbad, CA 92008 U.S.A  
Phone: +1.760.929.9972  
Fax: +1.760.929.9953  
www.bandb-medical.com

**EC REP** **EU Representative:**  
MDSS GmbH  
Schiffgraben 41  
30175 Hannover Germany  
Phone: +49 511 62628630  
www.mdssar.com





### READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE

1. Clean and dry the patient's skin and E.T. Tube application area. Be sure all moisture, oils and secretions are removed.
2. Apply the supplied skin preparation solution to both the skin and E.T. Tube application areas and fan dry for 25 seconds.
3. Peel backing from Part A. Secure the E.T. Tape II™ to the patients skin with the T-Tab directly over the desired E.T. Tube location (see Fig. A).
4. Wrap the short leg of the T-Tab around the E.T. Tube.
5. Tightly wrap the long leg of the T-Tab around the tube overlapping the short leg. This wrap must be tight as to compress the foam, eliminating any gaps which would allow secretions to get between the E.T. Tape II and the E.T. Tube (see Fig. B).
6. Wrap Cable Tie around T-tab, endotracheal tube and Bite Block #11040 or #11160 and pull tight. Cut off excess Cable Tie and place Blue Cap on end of Cable Tie.
7. Pass E.T. Tape II behind the patient's neck, under the earlobes.
8. Peel backing from Part C.
9. While holding Part A with finger, pull the E.T. Tape II taut, overlapping Part A with Part C (see Fig. C).
10. Firmly press the tape for proper adhesion (see Fig. D).
11. To remove Cable Tie, remove Blue Cap and release lock on Cable Tie (see Fig. F)
12. E.T. Tape II may be cut to proper length to fit smaller patients.
13. Periodically inspect E.T. Tape II™ for loss of adhesion on face or E.T. Tube. Replace immediately if this occurs.

**CAUTION:** Single Patient Use Only • Non Sterile  
• Disposable • DO NOT REPROCESS or REUSE

Spanish, German, French on reverse side

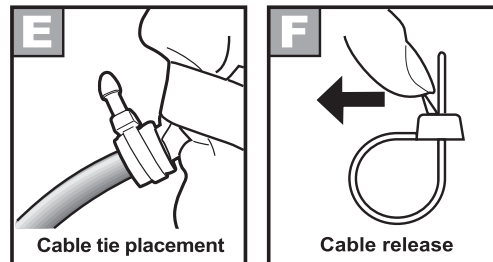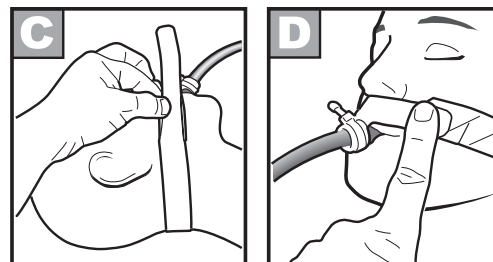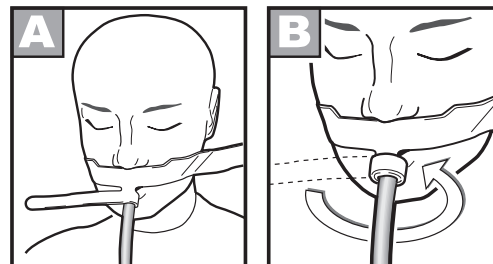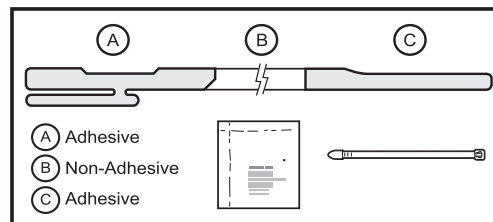Any serious incident that has occurred in relation to this device should be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established.

An instructional application video can be found at  
[www.bandb-medical.com/et-tape-ii](http://www.bandb-medical.com/et-tape-ii)

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