READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE

1. Secure the endotracheal tube per instructions or institution procedure. Gently pull the pilot tube in a straight line toward the end of the endotracheal tube.
2. Starting with the Bite-Proof Bite Block at a slight angle, snap the back of the Bite-Proof Bite Block onto the endotracheal tube. Keeping the Bite-Proof Bite Block at a slight angle, slide it onto the tube. Make SURE the pilot balloon is between the Bite-Proof Bite Block and the endotracheal tube (Figure B).
3. Keeping the pilot balloon tube taut, gently slide the Bite-Proof Bite Block down the tube until the wing is at lower lip.
4. Slowly rotate the Bite-Proof Bite Block until the wing is positioned laterally to the tube (Figure C). Peel backing from the wing and secure it to the cheek. Keeping the pilot balloon taut, position the Bite-Proof Bite Block to the proper depth (Figure D).
5. Secure the tab of the Bite-Proof Bite Block to the endotracheal tube with adhesive tape (Figure E). If using the B&B tube holders, secure tube according to direction and overlap tab with tape. Note: Tab must be securely taped to the endotracheal tube (Figure F).
6. Periodically inspect the Bite-Proof Bite Block for proper positioning and function. Replace immediately for loss of adhesion to the face or the E.T. Tube.
7. Bite-Proof Bite Block should be changed every two days for hygienic purposes.







CAUTION: NEVER use Benzoin or other adhesive-enhancing products in conjunction with this product. Should cuff filling problem occur, gently pull the pilot balloon tube taut to remove any kinks. NEVER remove the wing from the Bite Proof Bite Block.

CAUTION:

- Single Patient Use Only
- DO NOT REUSE
- DO NOT REPROCESS
- Not Sterile
- Disposable

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established.

An instructional application video can be found at www.bandb-medical.com/bite-proof-bite-block

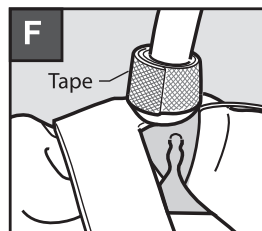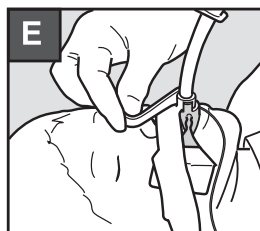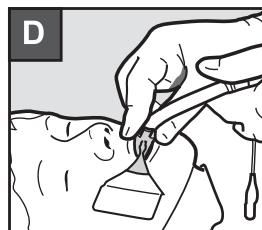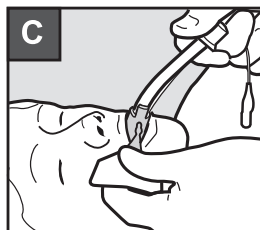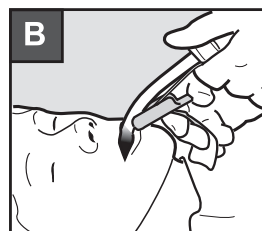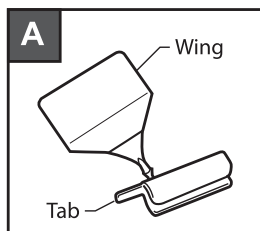B&B Medical Technologies®
www.bandb-medical.com

Bite-Proof Bite Block™

Latex-Free Endotracheal Tube Bite Block
For Tube Sizes 6 mm to 9.5mm

Contents: 1 UNIT

REF 11040

LOT

USE
BY:

32°C
90°F

Not Made With
Natural Rubber Latex

! READ PACKAGE
INSERT FOR
INSTRUCTIONS

2 — Disposable —
Single Patient Use Only
DO NOT REUSE

CAUTION: Federal law
restricts this device to sale by
or on order of a physician.

**NON
STERILE** Packaged Clean
— Not Sterile —

Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A.
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.bandb-medical.com

EC REP **EU Representative:**
MDSS GmbH
Schiffgraben 41
30175 Hannover Germany

Patented
Assembled in Mexico
Form: S104U Rev 1.4
© 2020 All Rights Reserved

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