StabilTube™

Latex-Free Long-Term
Endotracheal Tube Holder

Contents: 1 UNIT

REF 11060

LOT

USE
BY:

Packaged Clean
— Not Sterile —

— Disposable —
Single Patient Use Only
DO NOT REUSE

Not Made With
Natural Rubber Latex

Caution: Federal (USA) Law restricts this
device to sale by or on order of a physician.

Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.bandb-medical.com

EC REP

EU Representative:
MDSS GmbH
Schiffgraben 41
30175 Hannover Germany
Phone: +49 511 62628630
www.mdssar.com









READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE

1. Clean and dry the patient's skin and E.T. Tube application area. Be sure all moisture, oils and secretions are removed. Apply the supplied skin preparation solution to both the skin and E.T. Tube application areas and fan dry for 25 seconds. Release liner from Tee tab (Fig. A).
2. Place the retaining unit over the head of the endotracheal tube (Fig. B). Press the adhesive Tee Tab onto the ET Tube and secure the retaining unit tight, 1/2 inch above the lip line identification marking on the tube (Fig. C). Do not place anything between retaining unit and E.T. Tube.
3. Peel the remainder of backing from the face piece and secure to skin (Fig. D). Place over the lip for oral intubation (Fig. E) or over the bridge of the nose for nasotracheal intubation.
4. Slide the VELCRO® brand neckband under patient's neck and release hook and loop strap.
5. Thread neckband strap through the D ring and fasten. Unit should be snug around neck (Fig. F).
6. After confirmation of the proper tube placement, cut the excess tab from the Releasable Cable Tie 1/2 inch above retaining unit (Fig. G). Place Blue Cap on cut end of Cable Tie. Periodically inspect StabilTube for loss of adhesion on face or E.T. Tube and Cable Tie for secure retention. Replace immediately if either occurs.

To remove or reposition, remove Blue Cap, deflect Cable release and push excess tab through retaining unit (Fig. H)

CAUTION: NEVER use Tincture of Benzoin or other adhesive enhancing products with this product. Always use the Skin barrier provided.

- Single Patient Use Only • Non Sterile
- DO NOT REPROCESS or REUSE
- Disposable

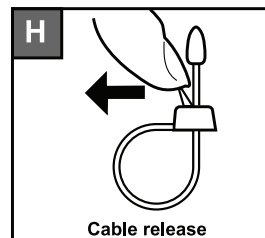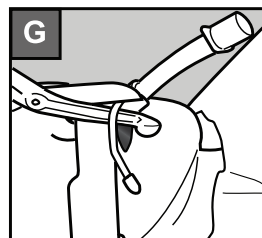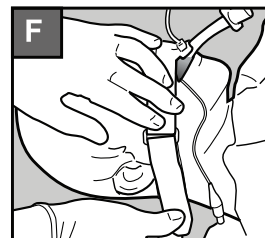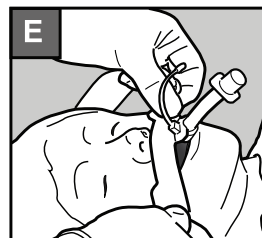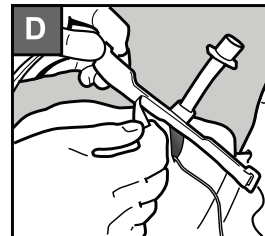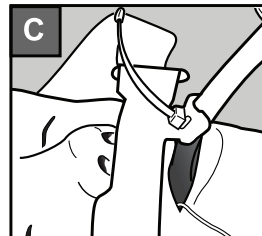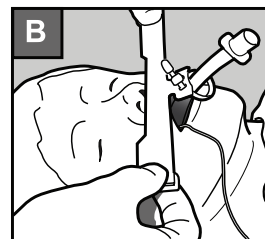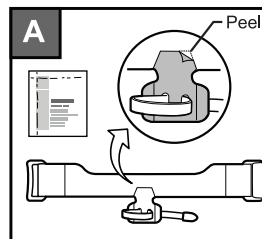An instructional application video can be found at:
www.bandb-medical.com/stabiltube

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established.

VELCRO® is a Registered Trademark of VELCRO Industries, B.V.

Spanish, German, French on reverse side.

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