
LockTite™

Non-Adhesive Endotracheal Tube Holder

Contents: 1 UNIT

REF 11090

LOT

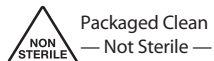Not Made With
Natural Rubber Latex

Caution: Federal (USA) Law restricts this device to sale by or on order of a physician.

Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.bandb-medical.com

EC REP **EU Representative:**
MDSS GmbH
Schiffgraben 41
30175 Hannover Germany
Phone: +49 511 62628630
www.mdssar.com

READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE

1. Place the retaining unit over the head of the endotracheal tube to the desired marking.
2. Secure the retaining unit tight, 1/2 inch above the identification marking on the tube.
3. Place VELCRO® brand neckband under the patient's neck, through the D ring, and fasten. Unit should be snug around neck.
4. After confirmation of the proper tube placement, cut the excess tab from cable tie. Periodically check for proper position. Change unit as necessary.
5. LockTite is specifically designed for use with the B&B UNIVERSAL BITE BLOCK™ (11160 Adult, 11170 Pediatric) and BITE-PROOF BITE BLOCK™ (11040). See instructions for use.

Cable Release:

To release, grasp the head of the cable tie and Endotracheal Tube, pull release and push excess tab through unit.

CAUTION: Federal (USA) law restricts this device to sale by or on order of a physician.

- Single Patient Use Only
- Disposable
- Not Sterile
- DO NOT REUSE

1

2


3

4

5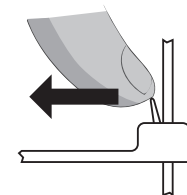

Cable Release

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established.

An instructional application video can be found at www.bandb-medical.com/locktite

VELCRO® is a registered trademark of VELCRO Industries, B.V.

Spanish, German, French on reverse side.

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VELCRO® ist ein eingetragenes Warenzeichen von VELCRO Industries, B.V.
VELCRO® é uma marca registrada da VELCRO Industries, B.V.