**READ AND UNDERSTAND
COMPLETE INSTRUCTIONS
PRIOR TO USE**

Use with B&B ET Tape II #11010, ET Tape #11020, StabilTube #11060 or other endotracheal tube securing devices.

1. Secure the endotracheal tube per instructions or institution procedure. Gently pull the pilot balloon tube in a straight line toward the end of the endotracheal tube.
2. Starting with the Universal Bite Block at a slight angle, snap the bite block onto the endotracheal tube. Keeping the bite block at a slight angle, slide it onto the tube. Make sure the pilot balloon tube is in between the Universal Bite Block and the endotracheal tube (see figure B).
3. Pull the Universal Bite Block up until it rests against the endotracheal tube securing tape and rotate the bite block to the lateral position (see figure C).
4. Remove Blue Safety Cap and place off to the side away from the patient. Wrap the elongated tab of the Cable Tie around the securing tape where it wraps around the endotracheal tube and feed through retaining unit (see figure D).
5. After confirming proper tube placement, pull Cable Tie through the retaining unit to secure to the endotracheal tube and carefully cut the releasable Cable Tie (straight across, not on an angle) 1/2 inch above retaining unit. Place Blue Safety Cap securely onto cut end of Cable Tie (see figure E).
6. To reposition or replace Universal Bite Block, carefully remove Blue Safety Cap from Cable Tie, release the locking tab and push excess Cable Tie through retaining unit (see figure F).

WARNING: If releasable Cable Tie is not used, Universal Bite Block must be securely taped to the endotracheal tube. If Cable Tie is not trimmed, remove and discard Blue Safety Cap in proper receptacle. Always place Blue Safety Cap off to the side when securing or repositioning Universal Bite Block. Should endotracheal tube cuff filling problems occur, gently pull pilot balloon tube taut to remove any kinks.


NOTE: Periodically inspect the Universal Bite Block to assure proper position and function of the bite block. Universal Bite Block should be changed every three days for hygienic purposes.

CAUTION:

Single Patient Use Only • Disposable • Not Sterile
• DO NOT REUSE • DO NOT REPROCESS

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established.

An instructional application video can be found at www.bandb-medical.com/universal-bite-block-adult/

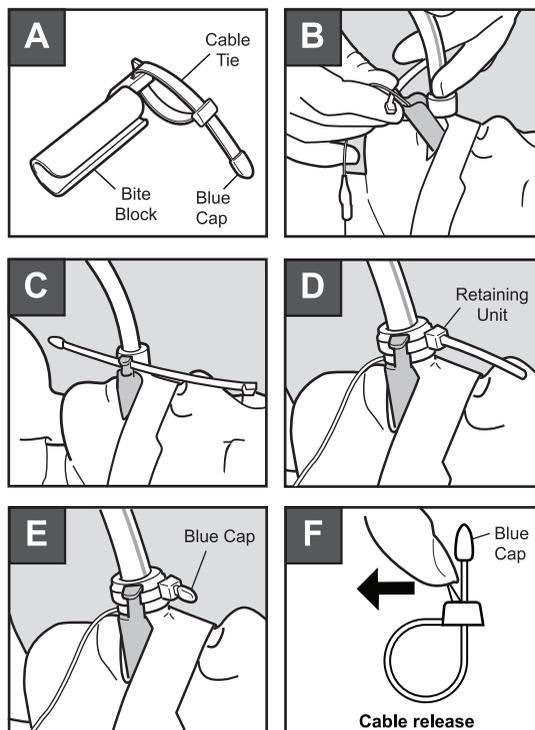BB Medical Technologies®
www.bandb-medical.com

Universal Bite Block™

Endotracheal Tube Bite Block
For Tube Sizes 6 mm to 9.5 mm

Contents: 1 UNIT

REF 11160

LOT

USE BY:

43°C
110°F

Not Made With
Natural Rubber Latex

! READ PACKAGE
INSERT FOR
INSTRUCTIONS

2 — Disposable —
Single Patient Use Only
DO NOT REUSE

CAUTION: Federal law
restricts this device to sale by
or on order of a physician.

NON STERILE
Packaged Clean
— Not Sterile —

Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.bandb-medical.com

EC REP **EU Representative:**
MDSS GmbH
Schiffgraben 41
30175 Hannover Germany

Patented
Assembled in Mexico
Form S116U Rev 1.6
© 2019 All rights reserved.

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