
READ AND UNDERSTAND COMPLETE INSTRUCTIONS PRIOR TO USE

Use with E.T. Tape or adhesive tape products.

1. Secure the endotracheal tube per instructions or procedure.
2. With the Universal Bite Block at a slight angle, slip the bottom portion of the device onto the endotracheal tube behind the tape.
3. Pull the Universal Bite Block forward until it rests against the tape.
4. Rotate bite block to lateral position.
5. Insert the elongated tab of the cable tie tightly around the tape where it circumvents the tube.
6. After tube placement confirmation, cut off remaining cable tie.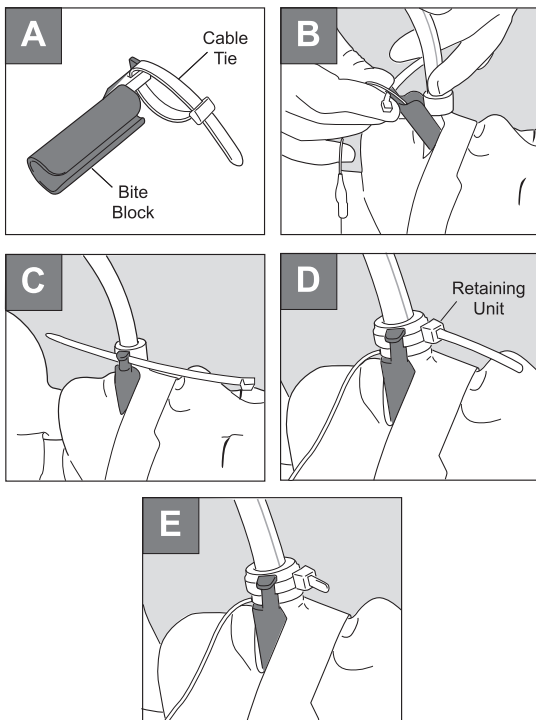

CAUTION: If the cable tie is not used, the tab must be securely taped to the endotracheal tube.

NOTE: Periodically inspect the bite block to assure proper position and function of the device.

The Universal Bite Block should be changed every three days for hygienic purposes.

CAUTION:

- Single Patient Use Only
- DO NOT REUSE
- Not Sterile
- Disposable

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established.

An instructional application video can be found at www.bandb-medical.com/content/11170_vid.htm

Pediatric Universal Bite Block™

Endotracheal Tube Bite Block
For Tube Sizes 4 mm to 6 mm

Contents: 1 UNIT

REF 11170

LOT

43°C
110°F

Not Made With
Natural Rubber Latex


— Disposable —
Single Patient Use Only
DO NOT REUSE

Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.BandB-Medical.com

EU Representative:
MDSS GmbH
Schiffgraben 41
30175 Hannover Germany

CAUTION: Federal law
restricts this device to sale by
or on order of a physician.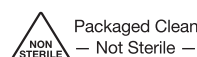

Packaged Clean
— Not Sterile —

Patented
Assembled in Mexico
Form S117U Rev 1.2
© 2020 All rights reserved.

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